




LISCLARE™
FREEDOM TO CARE




Accessory Instruction Manual

Safety side height extension, telescopic

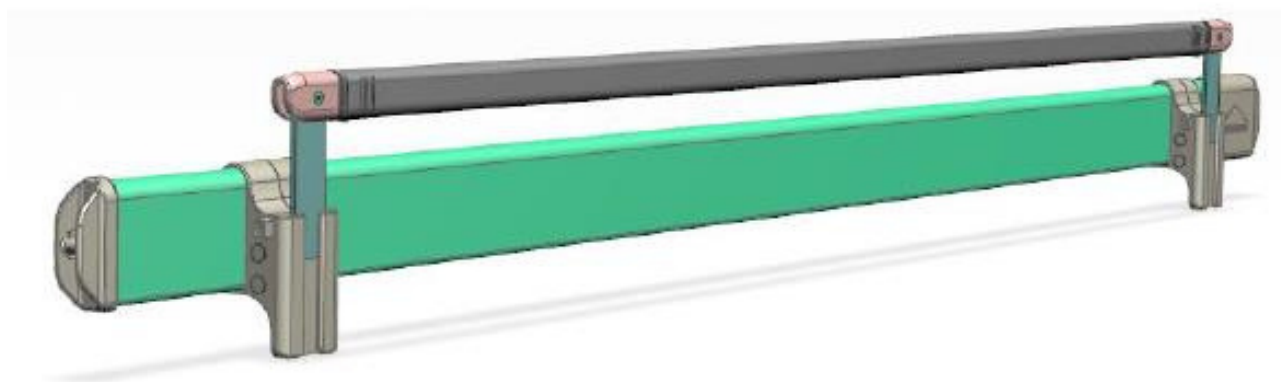

For attaching to:

Burmeier care beds with
2.0 m/2.2 m full-length wooden
safety side bars with “Easy
Click” quick fixing

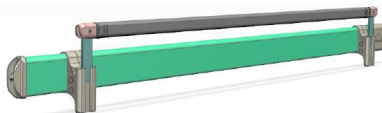	Version A: The attachable bar consists of one bar for attaching to <u>existing 2.0 and 2.2-m-long wooden bars</u>
	Version B: The attachable bar set consists of one bar that is <u>already attached to a new 2.0-m-long wooden bar</u>

General safety instructions



CAUTION

This instruction manual contains safety information and refers to potentially hazardous situations which, if not avoided, may lead to slight or minor injuries/hazards:

- **Observe your obligations as operator** in accordance with the Medical Device Regulation (EU) 2017/745 (MDR) and any existing additional relevant national laws/regulations, e.g. in Germany the Medical Devices Operator Ordinance (Medizinprodukte-Betreiberverordnung, German abbreviation: MPBetreibV), to ensure the long-term safe operation of this medical product with no risk of danger to patients, users or third parties.
- Instruct every user about hazards that may arise if this product is not used as intended.
- **As a user of this product, please observe** the contents of this instruction manual and follow the safety instructions.
- Continue to observe and follow the instruction manual for the beds for which the use of these accessories is allowed.
- Before using this product each time, make sure that it is in good working order and in proper condition.
- Clean and disinfect this product before using it for the first time.
- To avoid possible knocks and injuries when using the bed with the attachable bar installed, please note that the sides protrude outwards slightly.
- To avoid possible collisions and damage to property when moving the bed with the attachable bar installed, please note that the sides protrude outwards slightly.

Intended use

- This accessory increases the protective height of the safety sides on Burmeier care beds by 70 mm.
- This makes it possible to use thicker mattresses or positioning systems in these Burmeier beds and still ensure compliance with the required safety side protective height of 22 cm above these mattresses/positioning systems.

Use for the intended purpose

This accessory may only be used on Burmeier care beds that are prepared for this purpose and have all of the following features:

- safety sides comprising continuous wooden bars that are 2.0 m or 2.2 m long
- and
- safety sides equipped with the tool-free “Easy Click” quick fixing system (see adjacent illustration)




This accessory may only be used under the operating conditions described in this instruction manual. Its use for any other type of application is deemed to be contrary to the intended purpose.

Installation


Generally, 2 identical attachable bars are required for each bed. There are 2 versions:

Version A: Attachable bar for attaching to existing upper wooden bars

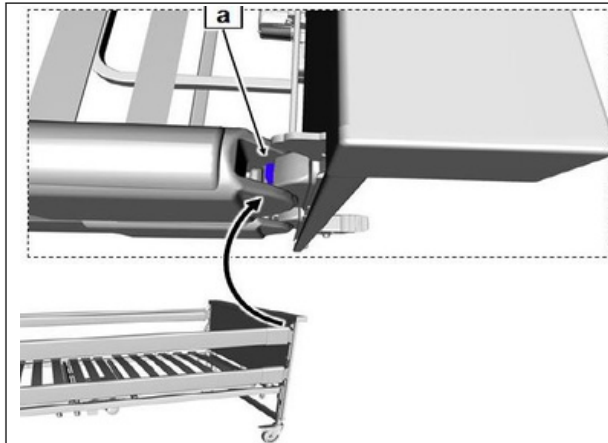


1. Remove the top bar

- Raise the safety side on the bed.
- Press the release button [a] in the safety side guide downwards with your finger, first at one end of the bar and then at the other end. Then lift out the top bar.

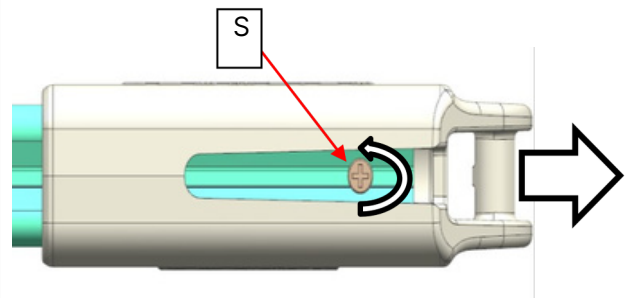


2. Remove the guide pocket

- Unscrew the limiting screw [S] on the underside of the bar and keep it safe for later use.
- Remove the guide pocket.

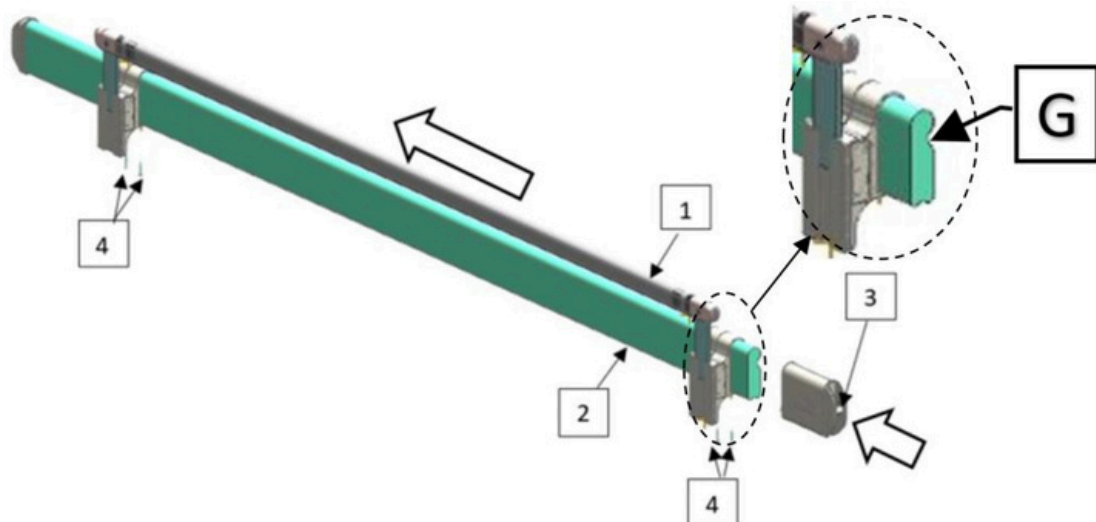

3. Slide the attachable bar [1] onto the wooden bar [2]



- The grip recess [G] of the wooden bar must be opposite the side of the protruding plastic holder.

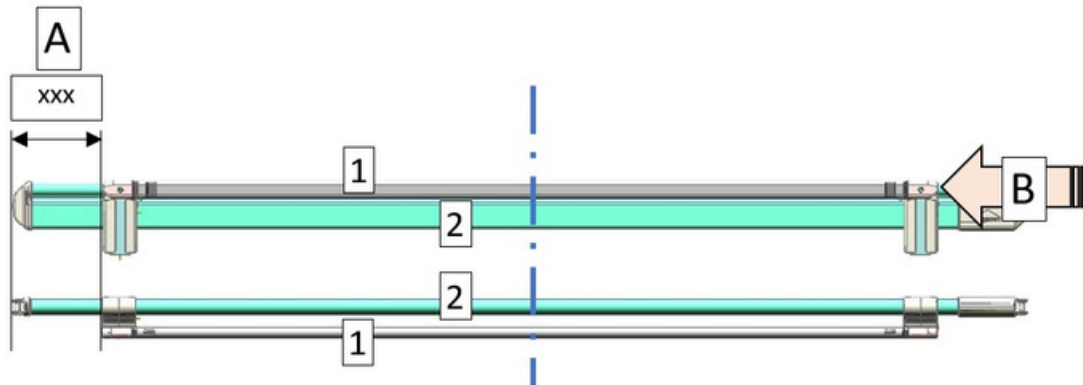

4: Position the attachable bar [1] "centrally" on the wooden bar [2]: Note the sequence! A: First measure the distance xxx from the end of the bar - then screw it tight there →see 5:

Wooden bar length	2.0 m	→ xxx = 175 mm
	2.2 m	→ xxx = 275 mm

B: Then push the attachment bar at the other end B in the direction of the already screwed-on holder A - then screw it on there →see 5:

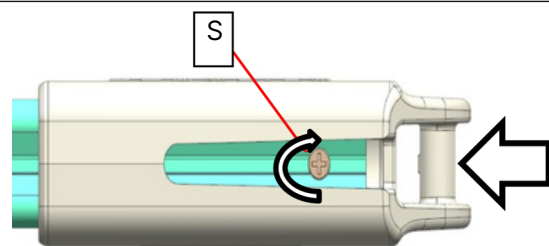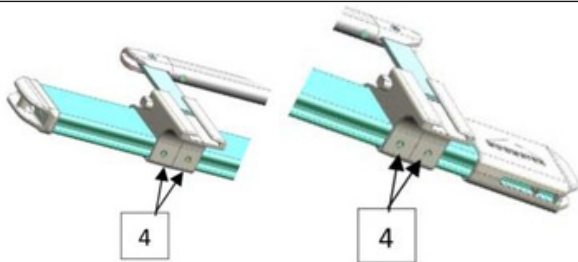


5: Fix the attachable bar [1] in place using the screws

- Use the 3.5x25 mm Spax screws [4] provided.
- Screw two screws [4] through the holes of each plastic holder and into the wooden bar.

6. Re-attach the guide pocket

- Take the previously removed limiting screw [S] and screw it into the underside of the wooden bar again.

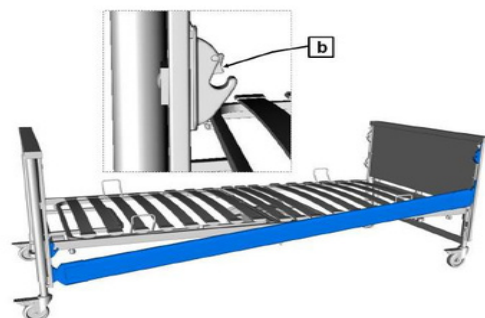

7. Re-attach the top wooden bar



The protruding attachable bar must face the outside of the bed – see illustration: top view of the bed

8. To fit the wooden bars, follow the instructions in the bed instruction manual

- Raise the guide rails at one end only – so that the bars are diagonal.
- On both sides of the bed, engage the bar ends one after the other into the holding devices a+b of the safety side guide rails.



The installation is complete.

- Make sure that attachable bar is securely fixed in place and cannot be pulled up or pushed along lengthwise.
- Check for smooth adjustment and secure latching in extended position Chap. "Operation" on following pages.
- Attach the second attachable bar on the other side of the bed following the same procedure.

Version B:





Attachable bar already attached to new wooden bars

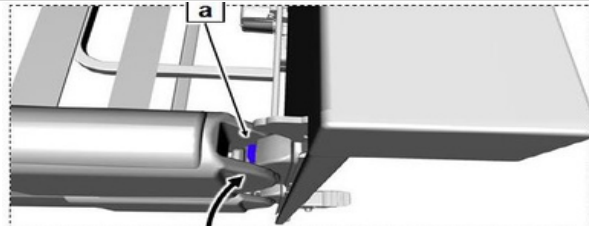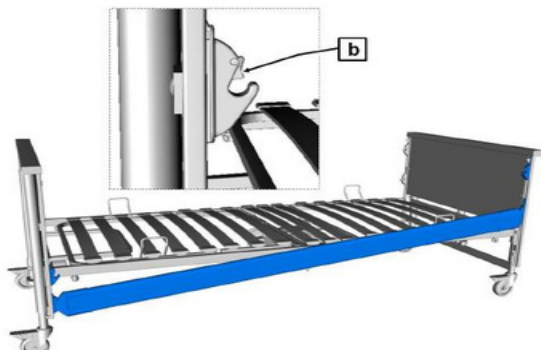	
1. Remove the old top bar • Raise the safety side on the bed. • Press the release button [a] in the safety side guide downwards with your finger, first at one end of the bar and then at the other end. Then lift out the top bar.	2. Install the new top bar with attachable bar  Attention! The protruding attachable bar must face the outside of the bed – see illustration: Top view of the bed
	
3. To fit the wooden bars, follow the instructions in the bed instruction manual • Raise the guide rails at one end only – so that the bars are diagonal. On both sides of the bed, engage the bar ends one after the other into the holding devices a+b of the safety side guide rails.	4. The installation is complete. • Make sure that the new bar is securely fixed in place and cannot be pulled up or pushed along lengthwise. • Attach the second attachable bar on the other side of the bed following the same procedure.

Operation

The following operating steps are to be carried out successively at each end of the attachable bar:

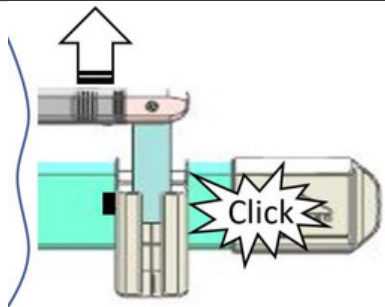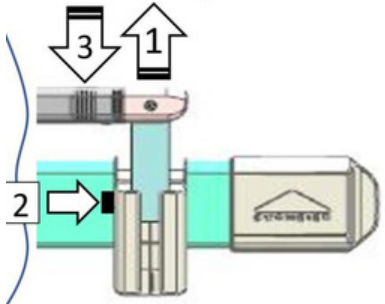	To raise the attachable bar • Pull up the attachable bar by hand until it engages with a "click". • Press down on the bar to check that it is securely locked in place.
	To lower the attachable bar 1. Pull up the bar with one hand and hold it there. 2. With the other hand, push the release button. 3. Lower the bar.

- The safety sides of the bed can still be operated with the attachable bar fitted.
-

CAUTION

Risk of entrapment/ risk of falling

Please also lower the top rail when the resident wants to get in/out of the bed.

- In this way, you enable comfortable, ergonomically optimal entry and exit.
- Please also observe the further instructions in the corresponding bed instructions for use.

Cleaning and disinfection

- The attachable bar can be cleaned and disinfected.
- Please also follow the further relevant information given in the instruction manual for your bed.

Maintenance

- The mechanical components responsible for securing the attachable bar must be inspected regularly for possible damage (recommendation: at least once a year), smooth adjustment, secure detent. and to ensure that they are functioning properly.
- Replace defective components immediately.
- The attachable bar is maintenance-free.

Technical data

Dimensions/ materials

Length	Approx. 165.5 cm	Bar material	Powder-coated steel
Additional protective height	Approx. + 7 cm	holder material	Plastic PA6

Classification

Conforms to Medical Device Regulation (EU) 2017/745 (MDR)		UMDNS code:	10-341 (safety side)
Class I medical product		EMDN code:	V08068001 (BED RAILS)

Disposal information

The product contains separable plastic, wood and metal components. Accordingly, it must be separated by type and sent to the recycling systems in accordance with local regulations.

Declaration of Conformity

We, Stieglmeyer GmbH & CO. KG, in our sole responsibility as manufacturer declare that the design of this product complies with the provisions of REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL of 5 April 2017 (MDR). It is classified as a class I medical device.

The respectively valid complete current declaration of conformity is available on request in our customer centre.

Type plate

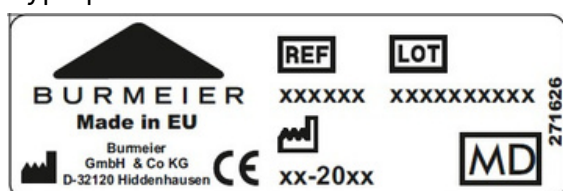



Illustration: example of type plate

Explanation labelling

	Item number		Order number
	Date of manufacture (week/year)		Medical device labels
	Attention! Follow the instruction manual		Conformity according to Medical Devices Regulation (EU) 2017/745 (MDR)

PID bar code

The additional PID bar code on the bed includes a number that clearly identifies each particular bed.

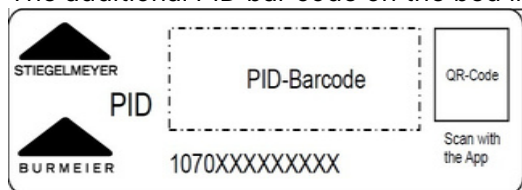


Manufacturer's address/service address

- In order to maintain operational reliability and the right to make claims under warranty, only original replacement parts must be used! To order replacement parts, or make customer service requests or
- other queries, please contact our customer centre:

Burmeier GmbH & Co. KG
Industriestraße 53
32120 Hiddenhausen, Germany
Phone: +49 (0) 5223 9769 - 0
Fax: +49 (0) 5223 9769 - 090
Email: info@burmeier.com
Website: www.Burmeier.com

Customers outside Germany can contact our distribution companies in their particular country if they have any questions. Contact details can be found on our website.

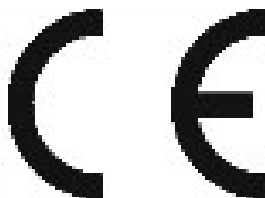

No part of this manual may be reproduced without the prior written permission of the publisher.

All rights reserved.

Subject to technical changes without notice!



Get in Touch

If you would like any further information, or to arrange a demonstration, please call:

0300 303 4790 (UK) | 01 633 2020 (IE)
lisclare.com | contact@lisclare.com

