


Small Manual Handling Aids

Manual Handling Positioning Wedge

When turning or repositioning patients it is important to fully support the patient. The wedge acts as a support for the patient to rest against, preventing them from moving in the bed. This means the carer doesn't have to hold the patient in place while working with them. The design of the wedge means the carer can use any of the three different angles. The outer material on the wedge can be wiped clean and the form inside is robust and will last the test of time.

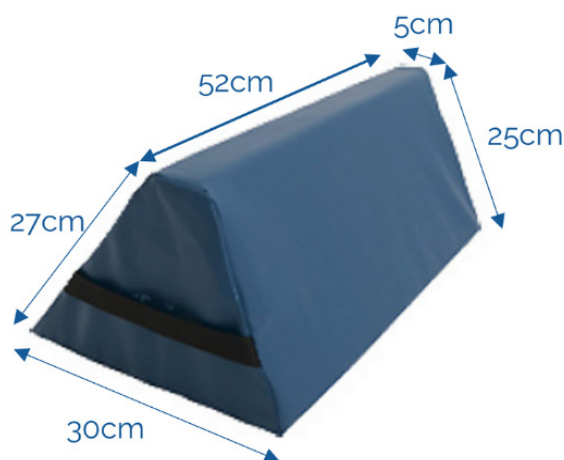

Material:

Positioning Wedge – Polyurethane Foam
Cover – Polyester, polyurethane

Washing Instructions:

Remove foam before washing. Do not add fabric conditioner. This product can be cleaned with a solvent-free cleaning agent with a pH level between 5-9, or a 70% disinfection solution.

Order code	Description	UOM	Min Vol Qty	Single	Volume
2002171	52cm Positioning wedge	EACH	4	65.00	54.00
2002172	82cm Positioning wedge	EACH	4	89.00	74.00

Checking:

The product should be checked regularly, preferably each time it is used and especially after washing. Check that there is no damage to seams or fabric.