

emotion Superfine® In-bed System



emotion®
move in comfort & safety

Move in Comfort & Safety

The emotion Superfine® In-bed system has been designed specifically for the community home care environment. Its key function is to facilitate turning, moving higher up in the bed and other transfers within the bed.

emotion Superfine® In-bed system is designed to stay permanently in bed - the fitted base sheet is placed on the bed with the smooth surface in the middle of the bed. The satin sliding base sheet can be combined with the Superfine® 4-way top sheet (with handles) for multi-directional sliding.

Maximise patient comfort and avoid bed sores.

Superfine® material can be placed on top of a pressure-relieving mattress without compromising the pressure-relieving properties of the mattress.

The material is designed to minimise friction and shearing damage to the skin, thereby reducing the risk of pressure sores.

The complete range of Superfine® In-bed system sheets come with hoist handles fitted.



The Superfine® base sheet is a satin sliding sheet available for two nursing bed sizes – 90 cm and 120cm wide beds. The sliding takes place sideways along the direction of the stripes


- Minimal manual handling effort required
- Reduced friction and shear, assists with profiling
- Weight tested up to 300kg
- Turning patients made easy

Click [HERE](#) to watch a video on our emotion Superfine In-Bed System

emotion Superfine® In-bed System

Facilitates turning in bed

The Superfine® in-bed system is intended for users who have difficulties turning in bed. The smooth and silky central part, enables turning with minimal effort. Sliding sideways is made easy without risk of sliding off the bed by the antislip material at both sides of the sheet. The material also prevents the user from sliding down in the bed. Superfine® fitted base sheets are placed on the bed like a normal sheet, with the smooth surface in the middle of the bed. For users who need more assistance, Superfine® can be combined with Superfine® 4-way sliding top sheet with hoist handles. The Superfine® Top Sheet Multidirectional drawsheet with handles is available in 2 sizes

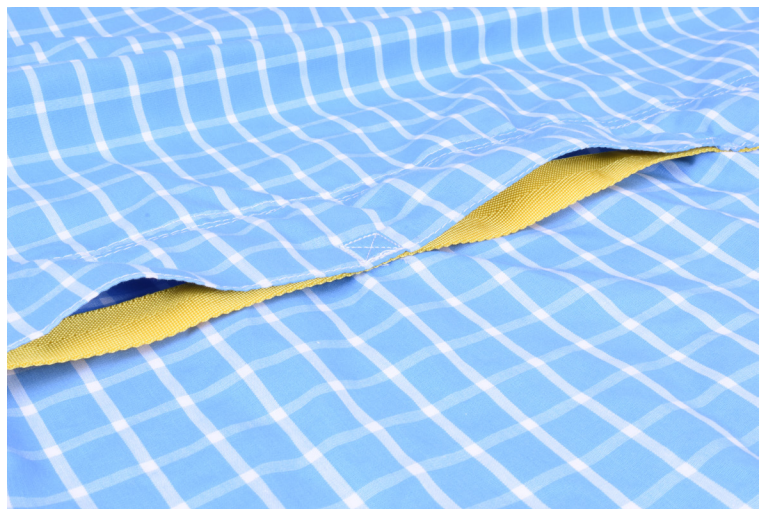

Easy and comfortable turning and positioning of users in bed

The Superfine® 4-way top sheet is placed on top of a Superfine® base sheet. A marking tag indicates which side should be facing upwards. The Superfine® 4-way has an antifriction material on the underside and the combination of these two Superfine® sheets allows for multidirectional moves in bed.

Procedures such as turning in bed, moving higher up/lower down in bed or re-adjusting a user while lying becomes simple to achieve with this combination, even with heavier or passive user. The function of Superfine® 4-way also makes it possible to perform gentle and comfortable transfers of users in pain. After turning, the gliding movement can be inhibited by tucking Superfine® 4-way in under the mattress on both sides of the bed. The Superfine® 4-way top sheet always comes with handles which allows the use of a looped overhead hoist and looped mobile hoist

Order Code	Description	UOM	Min Vol Qty	Single	Volume
2623002	e motion Superfine 4-way in-bed bari, c/w handles	SET	10	182.00	120.00
2623003	Superfine 4-way in-bed system TOP sheet 190Wx140L	EACH	10	82.00	54.00
2623005	Superfine 4-way in-bed system TOP sheet 190Wx200L	EACH	10	100.00	66.00
2623004	Superfine 4-way in-bed system BASE sheet 90Wx200L	EACH	10	76.00	51.00